

NASG Application at Community Level

Always wear gloves when putting the NASG on a bleeding woman.
One or two people can apply the leg segments (#1-3)



Place the NASG under the woman



If she can help, first have her raise her feet




Then place her feet on the bed or floor (not the NASG)



Have her raise her hips while you slide the NASG under her



Place the ball in segment #5 over her navel for fit, but do not close segments #5 and #6 until the end of the application

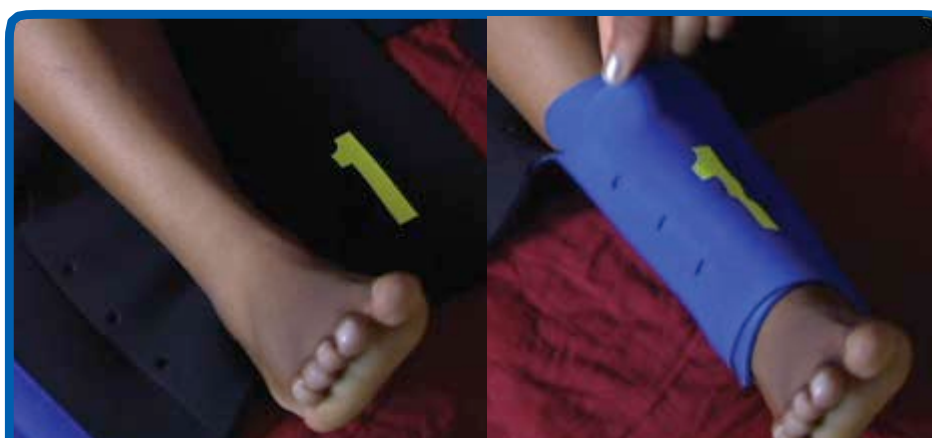

Start at the ankles, close segment pair #1 (or #2 for short women) around her ankles. Check for tightness by pulling up on each segment with your finger, let go, it should make a sharp sound



Close segment pair #2 as tightly as possible; snap each segment



Close segment pair #3 as tightly as possible; snap each segment



Only one person should close segments #4-6

Close segment pair #4 around the woman's public bone and pelvis



Place segment #5, with the ball, over the navel



Close segments #5 and #6 as tightly as possible



Ask the woman if she can breathe normally



Transfer the woman to a facility with blood and surgery

The NASG must only be removed by qualified skilled providers.

Only remove at facilities capable of providing blood and surgery and only after the woman's pulse and blood pressure have been stable for 2 hours and the cause of her bleeding identified and treated.